



Installation and Use Instructions

TL-5102MP Motion Sensor Caregiver Paging System

IMPORTANT INFORMATION: The TL-5102MP Motion Sensor Caregiver Paging System relies on wireless technology which is subject to physical and environmental considerations. These products do not have an out of range function and as such should be tested periodically in the setting in which they are to be used to understand their area of effective operation. Please be aware that the transmitter will not be 100% accurate if it is out of range at anytime. It is the end-user's responsibility to make sure that the product is used correctly and within range of each other. Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use. It is important to read and follow the directions.

WARNING: Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **This product is to be used by a licensed caregiver only. Test this device before each use. Read the instructions and Legal Disclaimer.**

The Smart Caregiver Corporation TL-5102MP is designed to be installed by the end user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

A.) Direct patient supervision B.) Adequate training for staff personnel for elopement and fall prevention

C.) Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of 90 days from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.



SMART CAREGIVER CORPORATION

1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637
sales@smartcaregivercorp.com | www.smartcaregivercorp.com

TL-5102MP Motion Sensor Caregiver Paging System



About The Motion Sensor Caregiver Paging System:

This simple system comes complete with one motion sensor and one Caregiver Pager. The Caregiver Pager has an audible alert and is small enough to be carried in a pocket or clipped to a belt. This Motion Sensor Paging System alerts the caregiver wearing the pager of a resident's activities when the resident interrupts the motion sensor's infrared field, as long as they are within 300 feet of each other. The Motion Sensor may be mounted next to a bed to indicate when a resident arises, or by a doorway to indicate when a resident is approaching the doorway. This system comes with a 90 Day Warranty.

TL-5102MP PACKAGE CONTENTS:

- 1 Caregiver Pager (batteries not included)
- 1 Motion Sensor (batteries not included)
- 2 Screws
- 2 Double Stick Tape Strips

Need Set-up Help? (800) 650-3637


Quick Start Instructions

Need Set-up Help? (800) 650-3637

TL-5102MP

1 Install Batteries into the Pager


Insert two new, AA batteries into the pager.

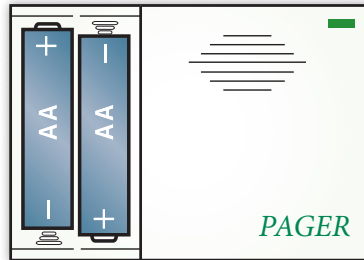

Note: Batteries are not included.

2 Install Batteries into the Motion Sensor


Insert three new AA batteries into the motion sensor.

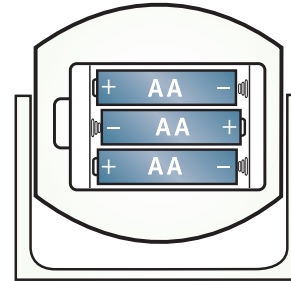

Note: Batteries are not included.


3 Select the Pager Alert Tone

Slide the switch on bottom of the pager to select a musical alert or a chime alert tone.

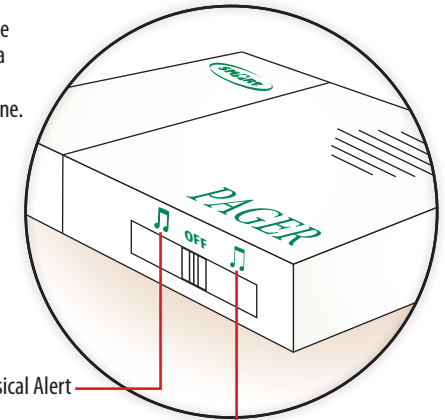


4 Turn on the Motion Sensor

Flip switch on front of the monitor to "On" position.

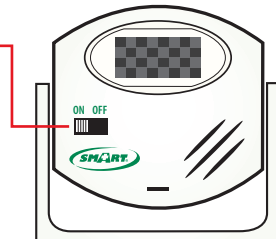


5 Test the System

Wave your hand over the front of the infrared sensor to test.

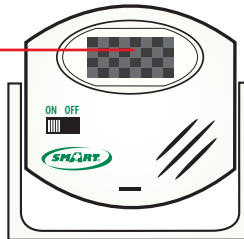


The pager will sound the alert tone when the infrared field is interrupted.

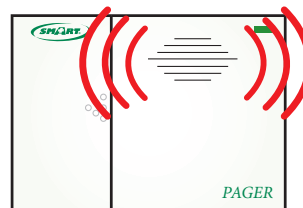


NOTE: THE END USER IS RESPONSIBLE FOR TESTING THE SYSTEM AT IT'S FURTHEST RANGE OF USE.

6 Mount the Motion Sensor

Headboard or wall mounted motion sensor

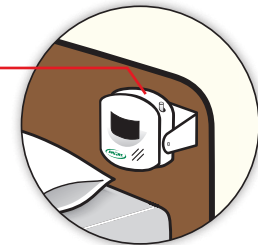


Double Stick Tape Mounting Bracket

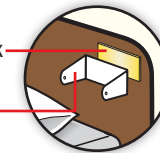

A. Wall Mount with Double Stick Tape

The motion sensor may be affixed to most smooth, clean surfaces with the included double stick tape.

Recessed Holes

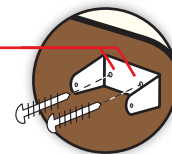

B. Wall Mount with Screws

The motion sensor may be more securely mounted to walls or headboards with the included screws. Use the recessed holes for screw placement on the motion sensor.

