

Caregiver Pager

433-PRB



433-CMU
Central Monitor Unit



433-PRB
Caregiver Pager with Reset Button & LCD Display

Blue Light = Activated
When light fades, replace
batteries.
"LB" will display

IMPORTANT INFORMATION: This Smart product uses wireless technology which is subject to physical and environmental considerations. This product does not have an out of range function and as such should be tested periodically in the setting in which it is used to verify its area of effective operation. Please be aware that the pager will not be 100% accurate if it is out of range at anytime.

Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use.

WARNING: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **TEST THIS DEVICE BEFORE EACH USE. READ THE INSTRUCTIONS AND LEGAL DISCLAIMER.**

The Smart Caregiver Corporation 433-PRB Pager is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A) Direct patient supervision
- B) Adequate training for staff personnel for fall prevention and elopement
- C) Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.

LIMITED WARRANTY: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of six months from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.



Need Set-up Help?

Call one of our friendly Customer Service Representatives at (800) 650-3637



1229 North McDowell Blvd, Petaluma, CA 94954
csr@smartcaregiver.com | www.smartcaregiver.com
Toll Free: (800) 650-3637

Caregiver Pager

433-PRB


- Numeric LCD Display
- Audible or Vibrate Options
- Built-In Reset Button

1 Set Up Pager

A. Install two "AA" batteries. Fresh batteries ensure best performance.

Re-Chargeable Batteries NOT Recommended

B. Turn pager on. Slide power switch to upper position for chime alert. Slide power switch to lower position for vibrate (quiet option).

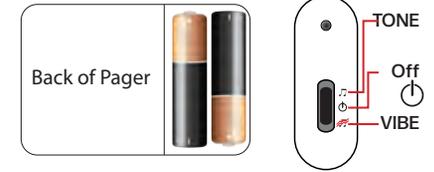

2 Set Up with 433-CMU Economy Central Monitor

To Program:

1. To add to Central Monitor with components already programmed:
 - Place the Central Monitor in the unlock position using switch on right side of monitor.
 - Press & hold the Program/Add New Components Button until you hear 3 tones (2 dings & melodic tone) then release button.
 - Once you hear the melodic tone, press the reset button on your 433-PRB Caregiver Pager.
 - Central Monitor will make a tone to confirm programming.
 - Press Reset button on bottom right corner of CMU.
 - Place Central Monitor back in the Locked position.
 - Test your system.




The 433-PRB Pager will automatically receive signals from any 433-CMU that is within range (up to 300* feet; test the system before each use). The LCD display on the pager will show, by number, which device triggered the alert. To use the Reset Button feature, pager must be programmed and will only reset the specific 433-CMU that it is programmed to.

2. Refer to the user instructions that came with your 433-CMU Central Monitor Unit for information about programming wireless sensor devices with the 433-CMU Central Monitor Unit.

3 Test the System - Central Monitor and Pager

The Pager and the Central Monitor should both alert when triggered by a sensor device. The Central Monitor and 433-PRB Pager will show the correct identification number of each device as it is tested. The Pager will only alert if within 300* feet from a Central Monitor. The 433-PRB Pager will respond to any 433-CMU Central Monitor within 300* feet. The Central Monitor and Pager will continue to alert until they are silenced.

To silence the Central Monitor, press the Reset Button on the 433-PRB Caregiver Pager or on the Central Monitor. This will silence the Central Monitor and Pager.

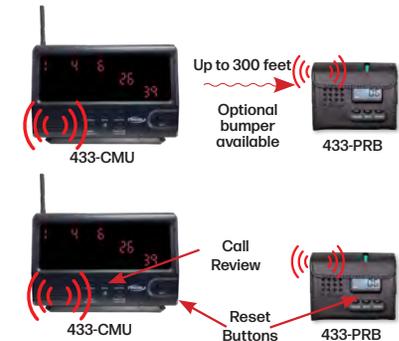

***IMPORTANT:** The end user is responsible for testing system at its furthest range of use. Maximum Range is 300 Feet.

Test system before each use.


433-PRB

Caregiver Pager

- Numeric LCD Display
- Built-In Reset Button
- Audible or Vibrate Options

1 Set Up Pager

- Install two "AA" batteries. Fresh batteries ensure best performance.
- Turn pager on. Slide power switch to upper position for chime alert. Slide power switch to lower position for vibrate (quiet option).

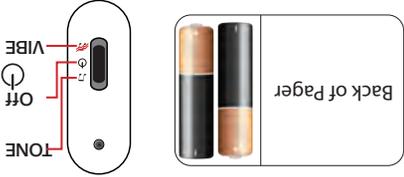

433-PRB

Caregiver Pager



IMPORTANT INFORMATION: This Smart product uses wireless technology which is subject to physical and environmental considerations. This product does not have an out of range function and as such should be tested periodically in the setting in which it is used to verify its area of effective operation. Please be aware that the pager will not be 100% accurate if it is out of range at anytime.

Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use.

WARNING: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **TEST THIS DEVICE BEFORE EACH USE. READ THE INSTRUCTIONS AND LEGAL DISCLAIMER.**

The Smart Caregiver Corporation 433-PRB Pager is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- Direct patient supervision
- Adequate training for staff personnel for fall prevention and elopement

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. THE SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed by a licensed caregiver only.

LIMITED WARRANTY: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of six months from the date of sale. Other than this limited warranty, seller makes no express or implied warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER, SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE. THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.



Need Set-up Help?
Call one of our friendly Customer Service Representatives at (800) 650-3637

1229 North McDowell Blvd, Petaluma, CA 94954
csr@smartcargiver.com | www.smartcargiver.com
Toll Free: (800) 650-3637



2 Set Up with 433-CMU Economy Central Monitor

The 433-PRB Pager will automatically receive signals from any 433-CMU that is within range (up to 300* feet; test the system before each use). The LCD display on the pager will show, by number, which device triggered the alert. To use the Reset Button feature, pager must be programmed and will only reset the specific 433-CMU that it is programmed to.

2. Refer to the user instructions that came with your 433-CMU Central Monitor Unit for information about programming wireless sensor devices with the 433-CMU Central Monitor Unit.


- Place the Central Monitor in the unlock position using switch on right side of monitor.
- Press & hold the Program/Add New Components Button until you hear 3 tones (2 dings & melodic tone) then release button.
- Once you hear the melodic tone, press the reset button on your 433-PRB Caregiver Pager.
- Central Monitor will make a tone to confirm programming.
- Press Reset button on bottom right corner of CMU.
- Place Central Monitor back in the Locked position.
- Test your system.



3 Test the System - Central Monitor and Pager

The Pager and the Central Monitor should both alert when triggered by a sensor device. The Central Monitor and 433-PRB Pager will show the correct identification number of each device as tested. The Pager will only alert if within 300* feet from a Central Monitor. The 433-PRB Pager will respond to any 433-CMU Central Monitor within 300* feet. The Central Monitor and Pager will continue to alert until they are silenced.

To silence the Central Monitor, press the Reset Button on the 433-PRB Caregiver Pager or on the Central Monitor. This will silence the Central Monitor and Pager.

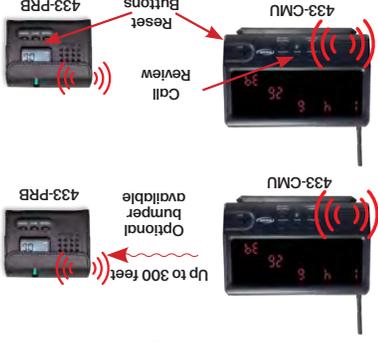

***IMPORTANT:** The end user is responsible for testing system at its furthest range of use. Maximum Range is 300 Feet. **Test system before each use.**