

SMART CAREGIVER CORPORATION



- Adjustable Volume
  - Low Battery Light
  - Integrated Mounting Clip
- ## Basic Pull-String Monitor

## Installation and Use Instructions

### BPS-01 Pull-String Monitor

**IMPORTANT INFORMATION:** Fall Monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all caregivers receive periodic training in the operation of these systems and that the systems are tested daily. It is important to read and follow directions.

**Read the instructions and Legal Disclaimer.**

The Smart Caregiver Corporation corded monitors are designed to be installed by the end user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A.) Direct patient supervision
- B.) Adequate training for staff personnel for fall prevention and elopement
- C.) Testing of the system before each use

**WARNING:** The manufacturer does not claim that this device will stop falls and/or elopement. This device is designed to augment caregivers' comprehensive resident mobility management program. Failure to comply with this warning may result in injury or death. This device is not suitable for all individuals. Other device may be required. This device is not a substitute for visual monitoring by a caregiver. Protective Boot may contain trace amount of chemicals known to the State of California to cause cancer and birth defects or other reproductive harm.


Where permissible, it is the seller's intent to limit any liability for special, consequential or incidental economic damages to refund of the purchase price or replacement of the goods. Seller disclaims any liability for compensatory or other damages arising out of the use of the goods.

**Limited Warranty:** Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, no express or implies warranties are given and no affirmation of seller, by words or action will constitute a warranty. The goods being purchased are being sold on a "withal faults" basis. The entire risk as to the quality and performance of the goods is with the buyer. Should the goods prove defective following their purchase, the buyer, and not the manufacturer, distributor or retailer, assumes the entire cost of all necessary installation, servicing or repairs.

### 1 Install Battery, Adjust Settings

Insert a new, 9 volt battery into the monitor.

**Set Alert Volume** The Volume is adjustable to be high or low

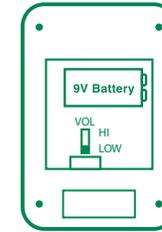


Back View of the Monitor Battery Compartment

### 2 Mount the Monitor

Attach the monitor to the handle or back of chair/wheelchair with the mounting clip provided. Make certain the monitor is securely attached and out of the resident's reach.



Mounted on Chair Handle

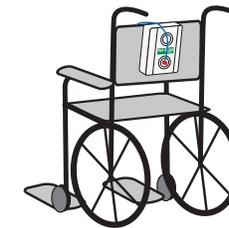

Mounted on Back of Chair

### 3 Test the System

With the monitor mounted on the chair, pull the string off the monitor. The monitor will sound the alert tone.



Mounted on Chair Handle



String Pulled and Monitor Alarming

### 4 Attach the Pull-String to Resident's Clothing

The string should be routed to allow the resident freedom of movement. Securely attach the string, out of reach, to the resident's clothing. Make sure the string is not obstructed by any objects, such as pillows or clothing.

When the resident moves up and out of the chair, the magnet will separate from the monitor triggering the alert.



Attach the String Clip to Resident's Clothing



# UPGRADE

## TO A WIRELESS PAGING SYSTEM



Does Not Startle The Patient



Pre-Paired and Easy-To-Use



Audible or Vibrate Alert



No Monthly Fees



Constant Monitoring



Pairs With Up To Six Sensors



### BED EXIT SENSOR

Alerts you when they get up from a bed



### CHAIR EXIT SENSOR

Alerts you when they get up from a chair



### FLOOR MAT SENSOR

Alerts you when they step on a floormat near a door or at the bedside



### CALL BUTTON SENSOR

Alerts you when they call for assistance



### DOOR EXIT SENSOR

Alerts you when they approach a doorway



### MOTION SENSOR

Alerts you when they get up from a bed or approach a doorway



## Claim Your Warranty

For Product Support and Coverage

## Ways to Register Your Product:



Visit [shop.smartcaregiver.com/warranty](http://shop.smartcaregiver.com/warranty)



Call Us! 1-800-650-3637



American Company  
& Customer Service Team

Questions? Call, Text or Email  
(800) 650-3637 | [csr@smartcaregiver.com](mailto:csr@smartcaregiver.com)