

# Installation and Use Instructions

## Weight Sensing Floor Mats

For use with TL-2100 Series Fall Monitors

- Great for monitoring bedsides and critical doorways
- SafeTRelease™ cord included



SMART CAREGIVER CORPORATION



## Floor Mat For TL-2100 Series Fall Mobility Monitors

**WARNING:** Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **Test this device before each use. Read the instructions and Legal Disclaimer.**

The Smart Caregiver Corporation Fall Monitor is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A.) Direct patient supervision
- B.) Adequate training for staff personnel for fall prevention and elopement
- C.) Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

**WARNING:** The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.

Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.



SMART CAREGIVER CORPORATION

1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637  
sales@smartcaregivercorp.com | www.smartcaregivercorp.com

## Pressure Sensing Floor Mats

Floor Mats alert caregivers whenever a resident steps onto the mat



### FLOOR MAT PLACEMENT

- Place the Floor Mat in the desired location such as a critical doorway or next to a bed.
- Place the monitor on headboard or wall
- Place the monitor out of the reach of the resident.



## Compatible with TL-2100 series fall /mobility monitors



TL-2100E

### Economy Fall Monitor

- Low Cost
- Universal Monitor for Bed, Chair Pads, Floor Mats and Chair Belts



TL-2100S™

### Safety Auto-Reset™ Fall Monitor

- One Button, Easy to use
- TamperProof™ with included Caregiver Key



TL-2100CP

### Change Pad Indicator™ Fall Monitor

- Avoid F-Tags
- One Button, Easy to use



TL-2100V

### Melody Fall Monitor with Recordable Voice

- Comes with Musical Alert
- Recordable Voice or Music

**Need Set-up Help? (800) 650-3637**


## 1 Install a 9 Volt Battery/Adjust Settings

The battery compartment door is secured with the included Phillips-head screw.

Note: Black ribbon goes underneath battery for easy removal.


### Optional Setting Adjustments

Refer to the user instructions included with your Smart Caregiver fall monitor for detailed set up information

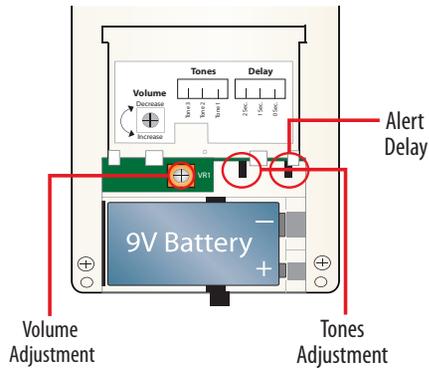


## 2 Connect the Floor Mat to the Fall Monitor

A. Pinch the cord together and route it through the strain relief recess.

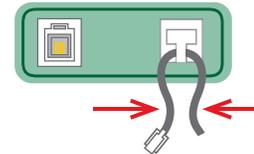


B. Plug the cord into the monitor.

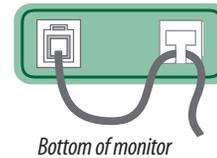

## 3 Test the System

Step on the floor mat. The alert will sound. Press the Reset button to silence and reset the monitor.

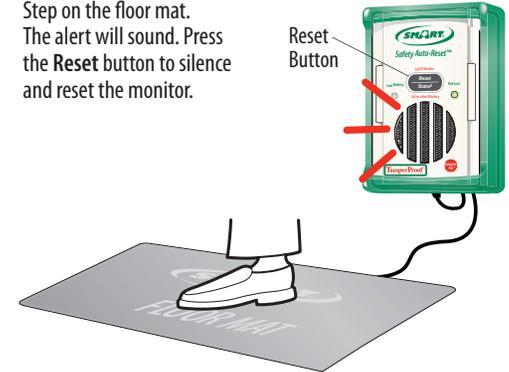

## 4 Mounting the Fall Monitor



1. Mounting screw in headboard or wall.
2. Hang the monitor on the screw using the mounting hole on back of the monitor.

Bed Use



Monitor may be mounted to a bed with the included locking mounting strap.

Doorway Use



## Resetting the Monitor/ Tamper Proof™ Mode with Caregiver Key™ (Optional)

Caregivers can choose to reset the Fall Monitor in either of two ways:


1. Pressing the Reset/Status button on the front of the monitor ("Reset Button ON" mode).
2. Placing the optional Caregiver Key™ over the red dot on the front of the monitor. ("Caregiver Key™ ON" mode).

*This mode allows only the caregiver to reset the monitor.*

Select either "Reset Button ON" or "Caregiver Key™ ON" with the switch on the side of the Fall Monitor.

**For more information about the Caregiver Key™ and how it can be used in your facility's Fall Assessment Program call 800-650-3637**

*TamperProof Mode not available on the TL-2100E Economy Fall Monitor.*

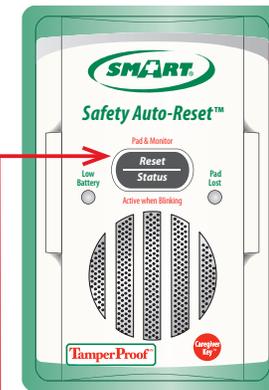

**"Reset Button ON" Mode**  
Press the Reset/Low Battery button to reset the monitor.



**"Caregiver Key™ ON" Mode**  
Place the Caregiver Key™ over the red dot to reset the monitor.