



Weight Sensing Landing Mat

- Great for monitoring at bedside or in doorway
- Available in CordLess® & Corded options

Installation and Use Instructions

Cordless & Corded® Weight Sensing Landing Mats

WARNING: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacture does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers comprehensive resident mobility monitoring program. Test this device before each use. [Read the instructions and Legal Disclaimer.](#)

The Smart Caregiver Corporation Corded & Cordless® Fall Prevention Monitors are designed to be installed by the end user. As such, it is the entire responsibility of the buyer to ensure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including but not limited to:

- A.) Direct patient supervision**
- B.) Adequate training for staff personal for fall prevention and elopement**
- C.) Testing of the system before each use**

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OF OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacture does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.


Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURE, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.



SMART CAREGIVER CORPORATION
 1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637
 csr@smartcaregiver.com | www.smartcaregiver.com

Weight Sensing Landing Mat Assembly Instructions

1. Place Landing Mat face down on a flat surface.
2. Peel off plastic layer on each of the Velcro squares on the sensor pad.
3. Place the sensor pad, Velcro side down on the bottom side of the landing mat. For CordLess Landing Mat, make sure the neck of the sensor pad is placed into the cutout area on the bottom side of the landing mat.
4. Press firmly on sensor pad to secure to landing mat.
5. If you are using a corded monitoring system, simply insert batteries into your monitor and plug in. Refer to your monitor instruction for further instructions if needed.
6. If you are using a CordLess monitoring system, please review the programming instructions that came with your CordLess monitor.

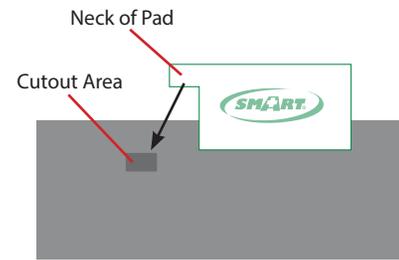

Cordless Setup



CordLess Setup



If you have any questions or concerns please feel free to call us.

Need Set-up Help? (800) 650-3637

CordLess® Landing Mat Instructions

1. Programming Landing Mats

The TL-2100G CordLess® Monitor may be used with CordLess® Landing Mats. To program a CordLess® Landing Mat:

1. Connect a CordLess® transmitter to a CordLess® Landing Mat (Figure 1).
2. Press and hold the Program Button on the back of the TL-2100G CordLess monitor until you hear 3 short beeps to enter programming mode. Release the button. The Status 1 LED light on the front of the monitor will remain lit for (30 seconds).
3. Turn on the CordLess® Transmitter that is connected to the Landing Mat. The red Status light on the transmitter will light up. (Figure 2)

For single pad use, press the reset button on the monitor to exit programming mode. The system is ready to use. Or.....

OPTIONAL: PROGRAM A SECOND PAD OF LANDING MAT

After programming Pad 1, the status 2 LED light will glow for 30 seconds, and the monitor will produce 2 short beeps, indicating that the monitor is ready to program a second landing mat or pad.

For pressure pads: Place pressure on the second pad to program to Pad 2. Release the pressure.

For a second landing mat, repeat step 3 above.

Press the Reset button to exit programming mode. The system is ready to use.

2. Landing Mat Operation

Important Note: In order for the CordLess® landing mat and monitor to fully reset, there must be a 15 second period with no pressure placed on the mat.

If pressure is placed on the landing mat before this 15 second period is complete, the monitor will produce a low volume alert and the Pad Lost light will blink. (This 15 second period allows a caregiver to assist a resident at a bedside while standing on the landing mat, but without the full volume alert continuing).

- During this 15 second period, the Reset button may be pressed again to silence the low volume alert and stop the Pad Lost light from blinking.
- After 15 seconds with no pressure on the landing mat the system will fully reset.

Test the system before each use:



1. Step on the landing mat
2. The alert will sound
3. Step off the landing mat
4. Press the Reset button to reset the system

3. Mounting the CordLess® Monitor

The TL-2100G CordLess Monitor may be placed up to 50 feet away from the pad. Mounting hardware is included for bed or wall mounting.

Note: Other mounting options are available.

Call 800-650-3637 for more information.

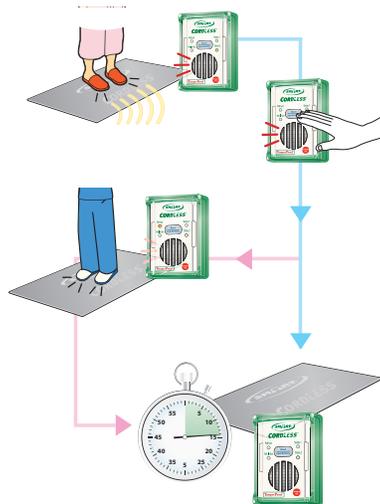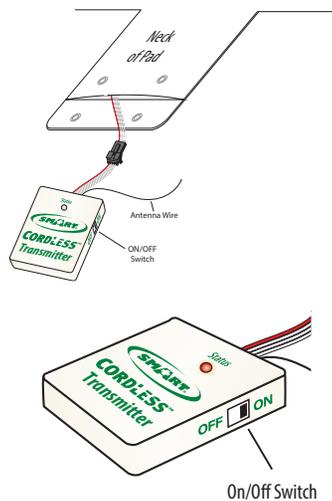


Corded Landing Mat Instructions

1. Install batteries into monitor

The battery compartment door is secured with the included Philips-head screw.

Optional Setting Adjustment

Refer to the user instructions included with your Smart Caregiver fall monitor for detailed setup information.

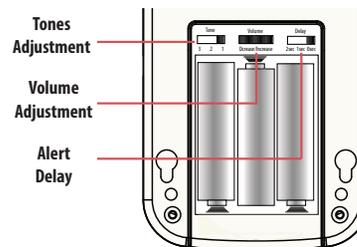


3. Test the system

Step on the landing mat. The alert will sound. Press the Reset button to silence and reset the monitor.




2. Connect the Landing Mat to the Fall Monitor

A. Pinch the cord together and route it through the strain relief recess.

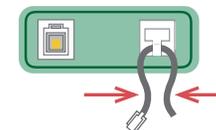

B. Plug the cord into the monitor

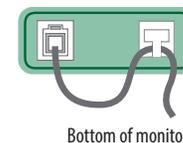

4. Mounting the fall monitor

1. Mount screws in headboard or wall.
2. Hang the monitor on the screw using the mounting hole on the back of the monitor.

