

SMART CAREGIVER Quick Start Instructions

THIS SYSTEM COMES PRE-PROGRAMMED OUT OF THE BOX. NO PROGRAMMING REQUIRED.

- 1 FOR FLOOR MATS ONLY:**
Open corner pocket of floor mat and pull out transmitter without unplugging it. (Fig. A)
- 2 TURN ON TRANSMITTER**
Push in **ON/OFF** button with paperclip or small pointed tool. (Fig. B)
- 3 RED STATUS LIGHT FLASHES**
When transmitter is on, **STATUS** light on front of transmitter will flash on and then turn off. (Fig. C)
- 4 INSERT BATTERIES** into pager or monitor. Set to tone/chime to turn on.

- 5 TEST THE SYSTEM**
Apply pressure to sensor pad/mat and release to test product before use. (Fig. D)
- 6 DO NOT TURN OFF TRANSMITTER**
After turning transmitter on, always keep on unless you need to store sensor pad/mat for over a month without use. Do **NOT** use as an ON/OFF switch.



Fig. A

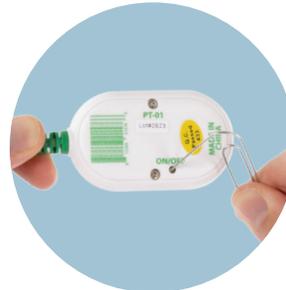


Fig. B

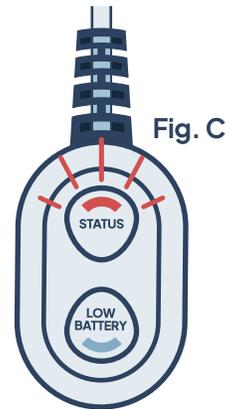


Fig. C

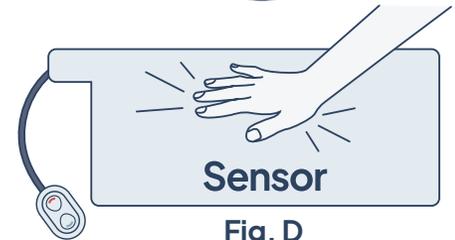


Fig. D



Need help?

Get expert live support from our local American team.

CALL / TEXT
800.650.3637
M-F, 7 AM-3:30 PM PT
smartcaregiver.com