

Caregiver Pager

TL-4016P



IMPORTANT INFORMATION: This Smart product uses wireless technology which is subject to physical and environmental considerations. This product does not have an out of range function and as such should be tested periodically in the setting in which it is used to verify its area of effective operation. Please be aware that the pager will not be 100% accurate if it is out of range at anytime.

Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use.

WARNING: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **TEST THIS DEVICE BEFORE EACH USE. READ THE INSTRUCTIONS AND LEGAL DISCLAIMER.**

The Smart Caregiver Corporation 433-PRB Pager is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A) Direct patient supervision
- B) Adequate training for staff personnel for fall prevention and elopement
- C) Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.

LIMITED WARRANTY: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of six months from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.



Need Set-up Help?

Call one of our friendly Customer Service Representatives at (800) 650-3637



1229 North McDowell Blvd, Petaluma, CA 94954
csr@smartcaregiver.com | www.smartcaregiver.com
Toll Free: (800) 650-3637

TL-4016P 4-2-21

Caregiver Pager

TL-4016P



IMPORTANT INFORMATION: This Smart product uses wireless technology which is subject to physical and environmental considerations. This product does not have an out of range function and as such should be tested periodically in the setting in which it is used to verify its area of effective operation. Please be aware that the pager will not be 100% accurate if it is out of range at anytime.

Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use.

WARNING: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **TEST THIS DEVICE BEFORE EACH USE. READ THE INSTRUCTIONS AND LEGAL DISCLAIMER.**

The Smart Caregiver Corporation 433-PRB Pager is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A) Direct patient supervision
- B) Adequate training for staff personnel for fall prevention and elopement
- C) Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.

LIMITED WARRANTY: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of six months from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.



Need Set-up Help?

Call one of our friendly Customer Service Representatives at (800) 650-3637



1229 North McDowell Blvd, Petaluma, CA 94954
csr@smartcaregiver.com | www.smartcaregiver.com
Toll Free: (800) 650-3637

TL-4016P 2-15-23

Caregiver Pager TL-4016P

- Numeric LCD Display
- Audible or Vibrate Options


1 Set Up Pager

A. "Unlock" Pager inside battery compartment. Install two "AA" batteries. Fresh batteries ensure best performance.

Re-Chargeable Batteries NOT Recommended

B. Turn pager on. Slide power switch to upper position for chime alert. Slide power switch to lower position for vibrate (quiet option).

C. With Pager set to "chime":
Select "Voice/Tone" by pressing button to desired tone.
Select "Volume" by pressing button to desired volume.
Once complete, switch to "Lock" position inside battery compartment.

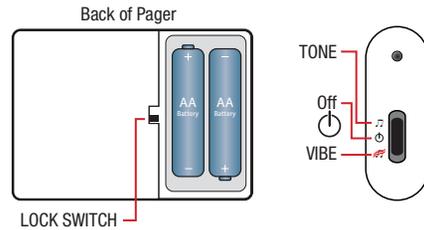

2 Functions:

A. "Review" button - Will display calls since pager turned on. When Pager turned off, calls are cleared.

B. When "Reset" button on TL-4016 monitor is pressed, Pager will be reset.

C. When using the TL-4016 monitor with a bed or chair pressure pad, if pressure reapplied to pad, Pager will automatically reset.


3 Test the System

The Pager and the TL-4016 monitor and/or TL-4015 Central Monitor should both alert when triggered by a sensor device. The Pager and the TL-4016 monitor and/or TL-4015 Central Monitor will show the correct identification number of device as it is tested. The Pager will only alert if within 300* feet from a TL-4016 monitor and/or TL-4015 Central Monitor. The monitor(s) and Pager will continue to alert until they are reset.



To silence the TL-4016 monitor, press the Reset button in upper left corner.

To silence the TL-4015 Central Monitor, reset component with corresponding reset button. This will automatically reset Pager as well.

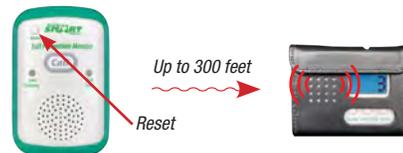

*IMPORTANT: The end user is responsible for testing system at its furthest range of use.

Maximum Range is 300 feet.
Test system before each use.

Caregiver Pager TL-4016P

- Numeric LCD Display
- Audible or Vibrate Options


1 Set Up Pager

A. "Unlock" Pager inside battery compartment. Install two "AA" batteries. Fresh batteries ensure best performance.

Re-Chargeable Batteries NOT Recommended

B. Turn pager on. Slide power switch to upper position for chime alert. Slide power switch to lower position for vibrate (quiet option).

C. With Pager set to "chime":
Select "Voice/Tone" by pressing button to desired tone.
Select "Volume" by pressing button to desired volume.
Once complete, switch to "Lock" position inside battery compartment.

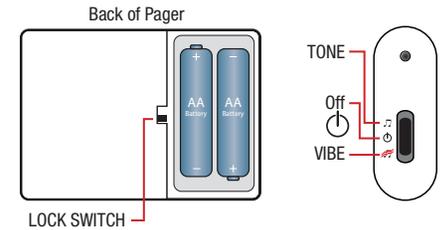

2 Functions:

A. "Review" button - Will display calls since pager turned on. When Pager turned off, calls are cleared.

B. When "Reset" button on TL-4016 monitor is pressed, Pager will be reset.

C. When using the TL-4016 monitor with a bed or chair pressure pad, if pressure reapplied to pad, Pager will automatically reset.


3 Test the System

The Pager and the TL-4016 monitor and/or TL-4015 Central Monitor should both alert when triggered by a sensor device. The Pager and the TL-4016 monitor and/or TL-4015 Central Monitor will show the correct identification number of device as it is tested. The Pager will only alert if within 300* feet from a TL-4016 monitor and/or TL-4015 Central Monitor. The monitor(s) and Pager will continue to alert until they are reset.



To silence the TL-4016 monitor, press the Reset button in upper left corner.

To silence the TL-4015 Central Monitor, reset component with corresponding reset button. This will automatically reset Pager as well.

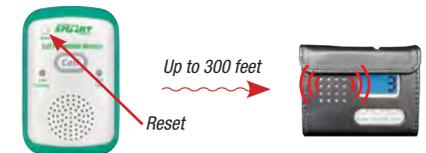

*IMPORTANT: The end user is responsible for testing system at its furthest range of use.

Maximum Range is 300 feet.
Test system before each use.