



## Installation and Use Instructions

### TL-5102MP Caregiver Pager With Bed Exit Alarm

**IMPORTANT INFORMATION:** Cordless and Wireless Systems rely on wireless technology which is subject to physical and environmental considerations. Please be aware that the transmitter will not be 100% accurate if it is out of range at any time. It is the end-user's responsibility to make sure that the product is used correctly and within range of each other. Cordless and wireless systems and devices are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and resident management practices. We recommend that all caregivers receive periodic training in the operation of these systems and that the devices are tested daily. It is important to read and follow the directions.

[Read the instructions and Legal Disclaimer.](#)

The Smart Caregiver Corporation devices are designed to be installed by the end user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A.) Direct resident supervision
- B.) Adequate training for staff personnel for fall prevention and elopement
- C.) Testing of the system before each use

**WARNING:** The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver.

Where permissible, it is the seller's intent to limit any liability for special, consequential or incidental economic damages to refund of the purchase price or replacement of the goods. Seller disclaims any liability for compensatory or other damages arising out of the use of the goods.

Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a warranty period from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty. No express or implied warranties are given and no affirmation of seller, by words or action will constitute a warranty. The goods being purchased are being sold on a "with all faults" basis. The entire risk as to the quality and performance of the goods is with the buyer. Should the goods prove defective following their purchase, the buyer, and not the manufacturer, distributor or retailer, assumes the entire cost of all necessary installation, servicing or repairs.



SMART CAREGIVER CORPORATION

1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637

sales@smartcaregiver.com | www.smartcaregiver.com



### About Caregiver Pager With Bed Exit Alarm:

This simple system comes complete with one Bed Exit Alarm and one Caregiver Pager. The Caregiver Pager has an audible alert and is small enough to be carried in a pocket or clipped to a belt. This Bed Exit Alarm Paging System alerts the caregiver wearing the pager of a resident's activities when the resident interrupts the Exit Alarm's infrared field, as long as they are within 150 feet of each other depending on the environment. The Exit Alarm may be mounted next to a bed to indicate when a resident arises, or by a doorway to indicate when a resident is approaching the doorway. This system comes with a 90 Day Warranty.

#### TL-5102MP PACKAGE CONTENTS:

- 1 Caregiver Pager (batteries included)
- 1 Bed Exit Alarm (batteries included)
- 2 Screws
- 2 Double Stick Tape Strips

**Need Set-up Help? (800) 650-3637**


## 1 Install Batteries into the Pager

Insert two new, AA batteries into the pager.



## 2 Install Batteries into the Bed Exit Alarm

Insert three new AA batteries into the motion sensor

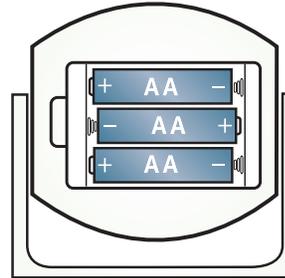

## 3 Select the Pager Alert Tone

Slide the switch on the side of the pager to select a chime tone or a alarm tone alert.



## 4 Turn on the Motion Sensor

Flip switch on front of the Bed Exit Alarm to "On" position.



## 5 Test the System

Wave your hand over the front of the infrared sensor to test.




The pager will sound the alert tone when the infrared field is interrupted.




## 6 Mount the Bed Exit Alarm

Headboard or wall mounted motion sensor

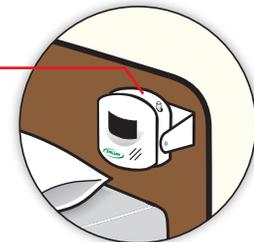


Double Stick Tape  
Mounting Bracket

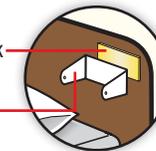

**A. Wall Mount with Double Stick Tape**  
The Bed Exit Alarm may be affixed to most smooth, clean surfaces with the included double stick tape.

Recessed Holes

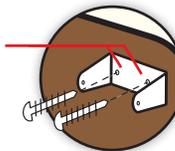

**B. Wall Mount with Screws**  
The Bed Exit Alarm may be more securely mounted to walls or headboards with the included screws. Use the recessed holes for screw placement on the motion sensor.

**NOTE: THE END USER IS RESPONSIBLE FOR TESTING THE SYSTEM AT IT'S FURTHEST RANGE OF USE.**