

Video Monitor for Caregivers



SMARTCAREGIVER.COM | CALL OR TEXT (800) 650-3637
AMERICAN COMPANY WITH AMERICAN CUSTOMER SERVICE



Video Monitor for Caregivers

WARNING

Please read the user guide carefully before using the device.

IMPORTANT INFORMATION: This system relies on wireless technology which is subject to physical and environmental consideration. These products should be tested periodically in the setting in which they are to be used to understand the area of effective operation. The visual and audio transmission will not be 100% accurate if it is out of range at any time. We recommend that all personnel receive periodic training in the operation of these systems.

WARNING: Failure to comply with the warning may result in injury or death, this device is not suitable for all individuals. Other devices may be required. Wireless systems and monitors are not a substitute for direct visual monitoring by a caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This system is designed as an adjunct to good caregiving practices to augment caregivers' comprehensive resident mobility management program. This system is not a substitute for proper staffing and patient management practices. Read the instructions and legal disclaimer. This wireless device is designed to be installed by the end user. As such, it is the entire responsibility of the buyer to verify that Smart Caregiver Corporation products are properly installed and tested successfully, before each use, in the setting in which they are to be used. Further, the system is not designed to replace good caregiving practices including, but not limited to: direct patient supervision; adequate training for staff personnel in fall management and elopement; and testing of the system before each use.

Camera settings and usage

Place the camera on a flat surface such as a table, desk or shelf. This monitor cannot be used instead of a caregiver. Keep this user guide in a safe place for future reference. Do not cover the monitor with a towel or blanket. Do not use the monitor near water. Do not install the monitor near a heat source. Only use the supplied charger and power adaptor. Never use another charger or power adaptor as this may damage the device. Do not touch the plug with a sharp or metal object

Important notes for installing a monitor

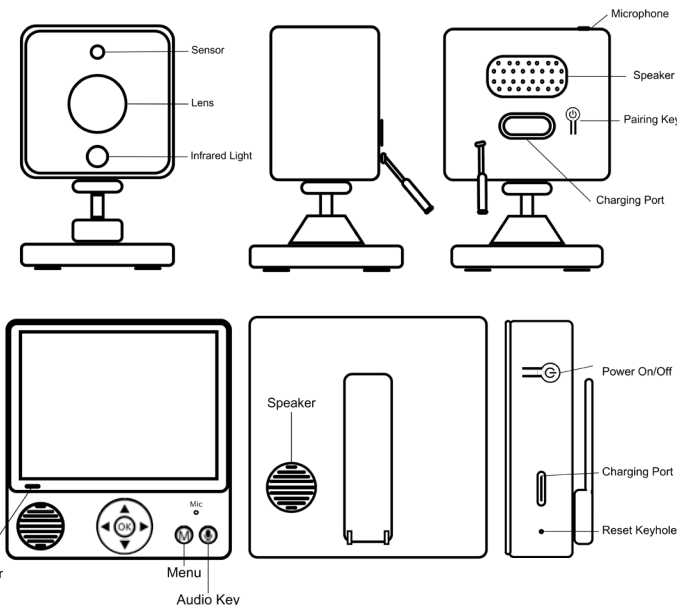
When using the camera and the display, you must be able to establish a wireless communication between them. The communication distance between the two devices is affected by environmental conditions. Any large metal objects between two devices may block wireless signals, such as refrigerators, metal doors or reinforced concrete. The strength of the signal may be attenuated by other solid buildings (such as walls), radios or electrical equipment (such as televisions, computers). If the signal lost, try to move the device to a different location indoors. Do not place the display and the camera in a range of 1.5 yards, otherwise it will cause audio feedback.

PRODUCT LIST

- (1) Camera
- (1) Monitor Display
- (1) User Manual
- (2) Power Adaptor
- (1) Charging Cable

DISPLAY ICONS

- SIGNAL LEVEL (5 LEVELS)
- NO SIGNAL
- CAMERA
- CAMERA IMAGE ZOOM 10X
- SIGNAL LEVEL (5 LEVELS)
- INTERCOM MODE
- NIGHT VISION
- SOUND DETECTION
- SYSTEM TIME SETTINGS
- BATTERY POWER
- LCD BRIGHTNESS
- CARE CHECK REMINDER



OPERATION

HOW TO TURN ON DEVICE

The monitor provides two power adaptors and charging cables. One is to power the camera, and the other is to charge the display's battery.

1. Connect the camera with power adaptor and plug into the 110-240V AC outlet. The camera will automatically be turned on, and the blue indicator will be on (the night mode will automatically turn off), Connect the display with power adaptor and plug into the 110-240V AC outlet.
2. Press and hold the power button of the display device for 3 seconds to turn it on or off.

PAIRING

Once the camera and the display are both turned on, they will be paired automatically. If the camera becomes unpaired with the display, follow the steps below to reconnect the camera with the monitor:

1. First, delete any previously paired cameras. Turn on the display and press M. Press < or > to select camera and press OK. In the pop-up menu, press to select and press OK. Select camera 1 , press OK, then select and press OK again to delete. Repeat for the other camera .
2. Press M. Press < or > to select camera and press OK. In the pop-up bar, press button to select and press OK to confirm, and then press or to choose or to pair. At this time, blue prompt will appear on the display screen.
If no device is paired for 30 seconds, it will automatically exit the pairing mode, or you can press and hold M for 1.5 seconds to exit the pairing.
3. When the camera is connected to the power supply and turned on, Press and hold the pairing button for 3 seconds, and then the blue light will flash quickly. The camera is now in the pairing mode. Once the camera has connected to the display, the indicator light will turn off. Now the display should show the image that the camera captures.




If there is no device pairing for 1 minute it will automatically exit the pairing mode.

(Note: One camera can only be connected to one monitor. If the pairing fails several times in a row, please contact us.)


Video Monitor for Caregivers - Instructions

CHECKING THE IMAGE OF THE CAMERA

When both the camera and the display are normally turned on, the display should be able to show the image from the camera (If the display does not show the image properly, please check the pairing condition.)

1. Check that the correct camera is selected. Press the **M** to enter the menu. Select the  camera, then select, * and ensure that  is selected. Please disregard the second camera icon as this monitor only comes with one camera.
2. If the connection fails, please re-pair according to the **PAIRING** instructions.

NIGHT VISION MODE




The camera is equipped with an infrared LED light, which helps to capture clear images in the dark. When the built-in sensor detects that the ambient light is weak, the infrared LED will automatically light up. The screen of the display will be shown in black and white, and the  icon will be displayed in the status bar at the top of the screen.

INSTALLATION

1. Place the camera in the area you want to monitor and plug in the camera using the included USB-C cable.
2. Adjust the camera view angle by turning the camera up, down, left or right until the desired image is shown on the display.


VIDEO MONITOR MENU

INTERCOM MODE

Press and hold the  key if you want to enter the two-way talk mode, and then you can talk with your loved one. Release the  key when the conversation is over. Meanwhile, the camera system will enter intercom mode after releasing the key  of the display so you can hear their response.

VOLUME ADJUSTMENT

There are two ways to adjust the volume.


1. In the main interface, short press the key **< or >** to adjust volume. There are 6 volume levels.
2. Or press the **M** to pop up the menu. Press the key **< or >** to choose , and press **OK** to confirm to enter the volume mode. Then press **< or >** button to adjust volume of the camera.

If the Video Monitor and Camera are brought within 5 feet of each other, you might experience audio feedback. If this occurs, lower the volume and move the Monitor away from the Camera.

VOX SETTINGS FOR BATTERY PRESERVATION



Turn on the VOX mode, and the display will turn off after 30 seconds when the camera side is in silent. When the ambient volume of the camera is louder than the set value, the display screen will turn back on.

1. Press **M** to enter the main menu.
2. Press the **< or >**, and press **OK** to enter **VOX** mode.
3. Select **LOW, MED, HIGH** sensitivity or press **OFF** button to turn off **VOX** mode.



Note: When the VOX mode is turned on, the screen and sound will be turned off after 30 seconds when the camera side is in silent. When the ambient sound of the camera is louder than the set value, the display screen will turn on and the speaker will make a sound. If VOX mode is OFF, you can short press  and the display will turn off. The sound from the camera can still be heard at any time.

DISPLAY BRIGHTNESS



There are two ways to adjust the display brightness.

1. In the main interface, press **▲** or **▼** to adjust the display brightness .
2. Press **M** to enter the main menu. Press **< or >** to choose , and press **OK**. Use arrow keys to adjust the brightness. There are 6 levels of brightness.



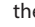
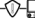
CAMERA ZOOM IN

Press **M** to enter main menu. Press **< or >**, and choose , and then press **OK** to zoom in and mark  will appear. Press **OK** again to recover the original image size.

POWER LEVEL

The mark  shows the current power level, The mark  is lower power level and will prompt low battery. After 5 consecutive beeps, the beep sound will disappear and only keep reminding **LOW BATTERY**.

SIGNAL

Signal mark . If there is a large signal tower base station or high power signal interference equipment in the current environment, it may cause interference to the product, and the mark  appears. The working distance between the camera and monitor is about 100 ft. but can be affected by walls, doors, or other interference. If the camera is unplugged or if the signal between the camera and monitor is lost for 5 seconds, the icon  will appear and flash, and the monitor will beep 5 times. After 5 beeps the monitor will leave the icon  flashing. Now the monitor is disconnected from the camera.


The camera and monitor should automatically reconnect when the camera is plugged back in or when the monitor is brought back in range of the camera. If not, please re-pair according to the PAIRING instructions.

CLEANING

Clean the device with a slightly damp cloth or an antistatic wipe. Never use detergents or abrasive solvents.

CARE CHECK REMINDER

Set a timer to remind you to do a care check-in

1. Press **M** to enter the main menu. Press **< or >** to choose , and press OK.
2. Select the timer duration and press OK.

**Questions? Call or Text our Customer Service Team
for Product Support (800) 650-3637**