



Installation and Use Instructions

TL-01 Non-Magnetic Pull-String Monitor

IMPORTANT INFORMATION: Systems and devices are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all caregivers receive periodic training in the operation of these systems and that the devices are tested daily. It is important to read and follow the directions.

Read the instructions and Legal Disclaimer.

The Smart Caregiver Corporation devices are designed to be installed by the end user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A.) Direct patient supervision
- B.) Adequate training for staff personnel for fall prevention and elopement
- C.) Testing of the system before each use

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver.

Where permissible, it is the seller's intent to limit any liability for special, consequential or incidental economic damages to refund of the purchase price or replacement of the goods. Seller disclaims any liability for compensatory or other damages arising out of the use of the goods.

Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a warranty period from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty. No express or implied warranties are given and no affirmation of seller, by words or action will constitute a warranty. The goods being purchased are being sold on a "with all faults" basis. The entire risk as to the quality and performance of the goods is with the buyer. Should the goods prove defective following their purchase, the buyer, and not the manufacturer, distributor or retailer, assumes the entire cost of all necessary installation, servicing or repairs.




SMART CAREGIVER CORPORATION


1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637
csr@smartcaregiver.com | www.smartcaregiver.com

11-15-18

TL-01 Non-Magnetic Pull-String Monitor

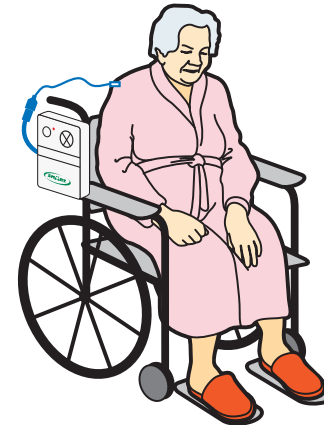

System in use, string attached to resident's clothing

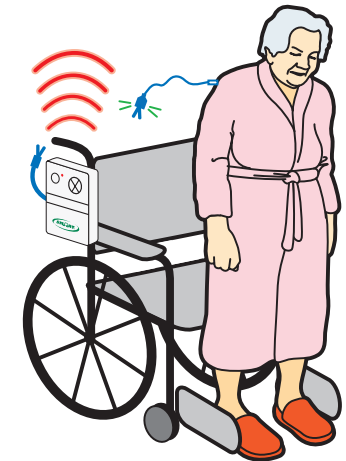

When the resident moves up and out of the chair, the separation switch will separate from the monitor, triggering the alert.

About The TL-01 Non-Magnetic Pull-String Monitor:

Pull-string monitors reduce falls by alerting staff when residents attempt to depart from their chairs. The TL-01 does not interfere with magnet sensitive equipment.


CONTENTS:

- Pull-String Monitor
- Mounting Strap
- Adjustable Pull-String Cord

Need Set-up Help? (800) 650-3637


1 Install the Battery

Insert a new, 9 volt battery into the monitor as shown.

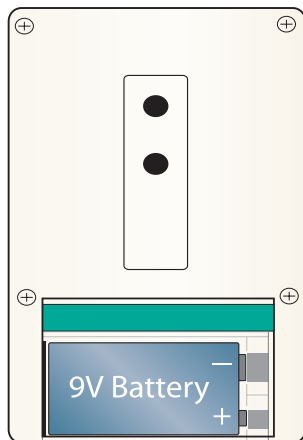

2 Mounting the Monitor

Attach the monitor to the handle or back of chair/wheelchair with the mounting clip and/or mounting strap provided. Make certain the monitor is securely attached and out of the resident's reach.

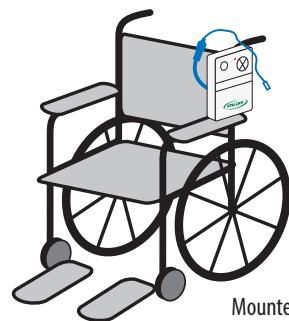


Mounted on
Chair Handle




Mounted on
Back of Chair

3 Test the System

With the monitor mounted on the chair, pull the string to separate the connection. The monitor will sound the alert tone.

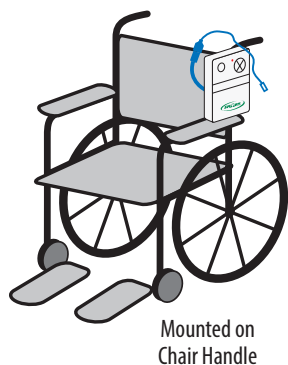

Mounted on
Chair Handle

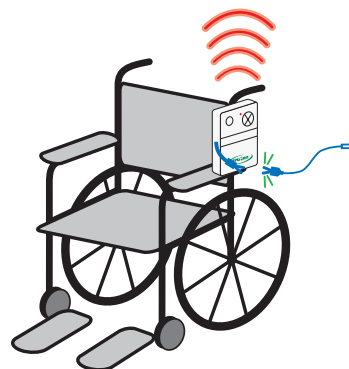


String Pulled and
Monitor Alarming

4 Attach the Pull-String to the Resident's Clothing

The string should be routed to allow the resident freedom of movement. Securely attach the string to the resident's clothing. Make sure the string is not obstructed by any objects, such as cushions or clothing.

When the resident moves up and out of the chair, the connector will separate from the monitor triggering the alert.

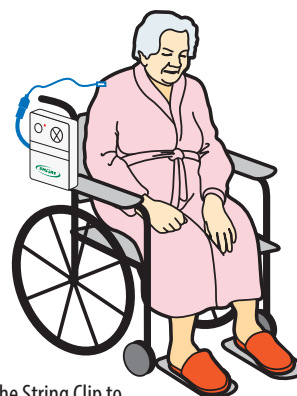

Attach the String Clip to
Residents Clothing

