



SMART CAREGIVER CORPORATION



- Integrated Mounting Clip
- TL-2000 includes life time warranty casing

TL-02/TL-2000 Pull-String Monitor

Installation and Use Instructions

TL-02/TL-2000 Pull-String Monitor

IMPORTANT INFORMATION: Systems and devices are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all caregivers receive periodic training in the operation of these systems and that the devices are tested daily. It is important to read and follow the directions.

Read the instructions and Legal Disclaimer.

The Smart Caregiver Corporation devices are designed to be installed by the end user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- A.) Direct patient supervision
- B.) Adequate training for staff personnel for fall prevention and elopement
- C.) Testing of the system before each use

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver.

Where permissible, it is the seller's intent to limit any liability for special, consequential or incidental economic damages to refund of the purchase price or replacement of the goods. Seller disclaims any liability for compensatory or other damages arising out of the use of the goods.

Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a warranty period from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty. No express or implied warranties are given and no affirmation of seller, by words or action will constitute a warranty. The goods being purchased are being sold on a "with all faults" basis. The entire risk as to the quality and performance of the goods is with the buyer. Should the goods prove defective following their purchase, the buyer, and not the manufacturer, distributor or retailer, assumes the entire cost of all necessary installation, servicing or repairs.



SMART CAREGIVER CORPORATION

1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637
 csr@smartcaregiver.com | www.smartcaregiver.com


Quick Start Instructions

1 Install Battery, Adjust Settings

Insert a new, 9 volt battery into the monitor.

Set Alert Volume

The Volume is adjustable to be high or low

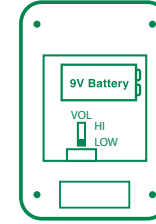


Back View of the Monitor Battery Compartment

2 Mount the Monitor

Attach the monitor to the handle or back of chair/wheelchair with the mounting clip provided. Make certain the monitor is securely attached and out of the resident's reach.




Mounted on Chair Handle

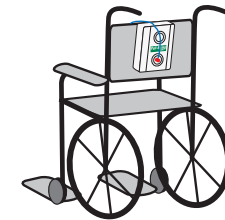


Mounted on Back of Chair

3 Test the System

With the monitor mounted on the chair, pull the string off the monitor. The monitor will sound the alert tone.

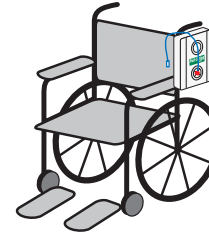

Mounted on Chair Handle

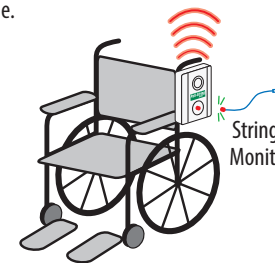


String Pulled and Monitor Alarming


4 Attach the Pull-String to Resident's Clothing

The string should be routed to allow the resident freedom of movement. Securely attach the string, out of reach, to the resident's clothing. Make sure the string is not obstructed by any objects, such as pillows or clothing.

When the resident moves up and out of the chair, the magnet will separate from the monitor triggering the alert.

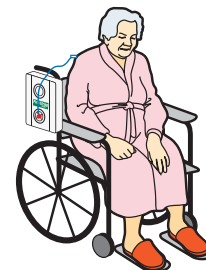

Attach the String Clip to Resident's Clothing

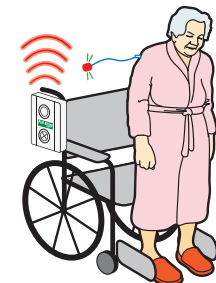

Products Better Than a Pull-String Monitor So They Don't Walk Alone

Economy WireLess Fall Prevention Alert



- ✓ Easy set-up and operation
- ✓ Quick set-up instructions included!
- ✓ For use with up to (6) wireless components (i.e. Call Buttons & Weight-Sensing Pads)

433-EC

WireLess Economy Quiet Fall Prevention Alert to Pager

- ✓ 2 ways to use
 - Call Button on front of monitor
 - Corded bed pad, chair pad, & floor mat systems also available
- ✓ Monitor sends wireless signal to pager for noise-free caregiver notification
- ✓ No programming required
- ✓ Easy as 1,2,3 set up




TL-2020

Call Us Today!
Let us help you
create a system
for your needs.

Facility Options Available to Match Your Needs WireLess Central Monitor

- ✓ Wireless so you can eliminate in-room alarm noise
- ✓ Flexible, easy-to-use central monitoring unit (Use one or up to 60 components)
- ✓ Alarms audibly & visually at central location and portable caregiver pager(s)
- ✓ Display indicates which resident triggered alarm
- ✓ Call ahead & have your system pre-programmed upon delivery

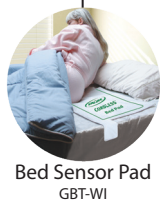

Bed Sensor Pad
GBT-WI



Floor Sensor Mat
FMT-07C



Wireless Call Button
433-NC

*additional options available

Low Cost Paging Systems



Motion Sensor to Pager
TL-5102MP



2 Push Buttons to Pager
TL-5102TP

Scan here to check out
our other great systems!

