

**TL-4015 Smart Station**

# Wireless Central Monitoring Unit



Caregiver Pocket Pager  
TL-4016P



DepartAlert® Door System  
TL-4005SYS

Reset Button  
4017-RB



Emergency Call Alert  
4017-ECA



Call Button  
with Emergency  
Pull Cord  
4017-CB



Quiet Wireless Monitor  
TL-4016R



**SMART CAREGIVER CORPORATION**

1229 North McDowell Blvd., Petaluma, CA 94954 | Tel: 800-650-3637  
csr@smartcaregiver.com | www.smartcaregiver.com

# Table of Contents

**\*This page is not correct, Table of Contents will be made after edits\***

<b>Features</b>	<b>Page 4-5</b>
<b>Pre-Programmed Instructions</b>	<b>Page 5</b>
<ul style="list-style-type: none"><li>• <b>*Pre-Programmed means - components are already programmed to CMU.</b></li><li>• You received your CMU with components already programmed.</li></ul>	
<b>Setup - Programming</b>	<b>Page 6</b>
<ul style="list-style-type: none"><li>• You received your CMU with NO programming.</li></ul>	
<b>Detailed Instructions For:</b>	<b>Page 7-9</b>
<ul style="list-style-type: none"><li>• TL-2020 Wireless Economy Fall Prevention Monitor</li><li>• 433-NC Nurse Call Button</li><li>• 433-TB Touch Call Button</li><li>• CordLess Weight Sensing Bed &amp; Chair Pads</li><li>• CordLess Weight Sensing Floor Mats</li><li>• 433-MS Motion Sensor</li><li>• 433-EXT Window/Door Exit Alarm</li><li>• 433-NCA Nurse Call Adapter</li><li>• 433-RB Caregiver Reset Button</li><li>• 433-PRB Caregiver Pager w/Reset Button</li><li>• 433-PGD Caregiver Pager</li></ul>	
<b>Resetting The Central Monitor</b>	<b>Page 10</b>
<b>Adding Additional Components</b>	<b>Page 11</b>
<ul style="list-style-type: none"><li>• You have a CMU with existing programmed components.</li><li>• You are adding additional components to it.</li></ul>	
<b>Replacing Existing Components</b>	<b>Page 11</b>
<ul style="list-style-type: none"><li>• You have a CMU with existing programmed components.</li><li>• You are replacing an existing programmed number with a new or different component</li></ul>	
<b>Mounting The Monitor</b>	<b>Page 12</b>
<b>Clearing The Monitor</b>	<b>Page 12</b>
<ul style="list-style-type: none"><li>• You need to clear out all programming to reprogram</li></ul>	
<b>Troubleshooting</b>	<b>Page 13</b>
<b>Warnings</b>	<b>Page 14</b>
<b>Legend</b>	<b>Page 15</b>
<b>Notes/Contact</b>	<b>Page 16</b>

# Mission & Value Statement

## Smart Caregiver Mission Statement

To ensure the safety and comfort of each and every patient and resident, Smart Caregiver is committed to supplying the least expensive and best quality patient monitors so that Care Givers can afford to have the necessary amount of monitors needed to keep people safe from falls.



Tim Long, Smart Caregiver President


**We are here to help!**

**Please call our world-class customer service department if you have any product questions or need help with the set up of your Central Monitor (800)650-3637.**




# Features


1. Quit
2. Program Button
3. ↓
4. ↑
5. Low Battery Indicator
6. Delete
7. Review
8. On/Off Switch
9. Antenna Port
10. Volume Control
11. AC Adapter Input
12. Antenna
13. Hook for Mounting Bracket
14. Battery Compartment
15. Base/Mounting Bracket

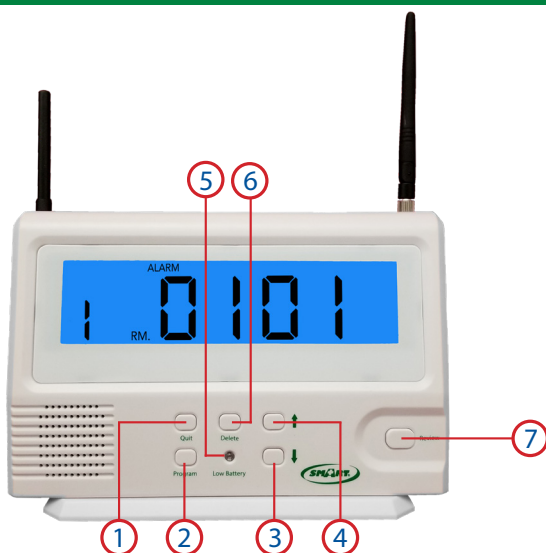

Front

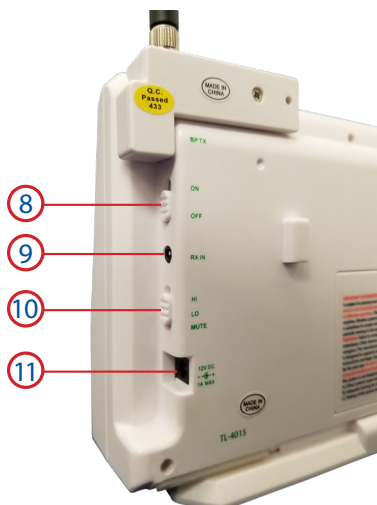

Side

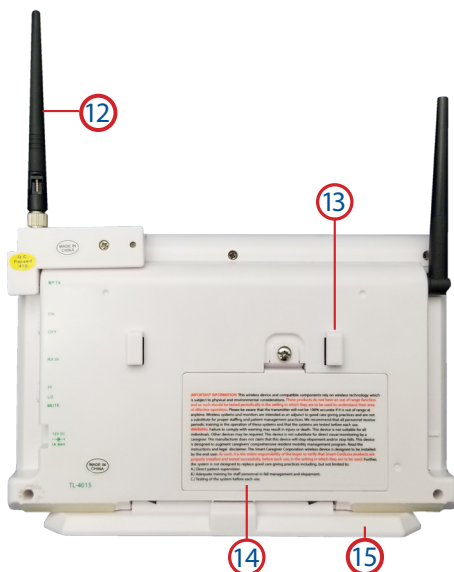



Back

# Features Explained



## 1-4 Manufacture Use Only: DO NOT USE

1. **Quit** - Only used by manufacturer for programming before shipping. DO NOT USE. Call Smart Caregiver at 800.650.3637 with any programming questions.
2. **Program Button** - Only used by manufacturer before shipping. DO NOT USE. Call Smart Caregiver at 800.650.3637 with any programming questions.
3.  - Only used by manufacturer for programming before shipping. DO NOT USE. Call Smart Caregiver at 800.650.3637 with any programming questions.
4.  - Only used by manufacturer for programming before shipping. DO NOT USE. Call Smart Caregiver at 800.650.3637 with any programming questions.



## 5-15 Customer Use Features

5. **Low Battery Indicator Light** - If using only batteries and they are low, indicator light will be red. When using AC adapter, indicator light will be green.
6. **Delete Button** - Used to delete/clear all calls received.
7. **Review Button** - Used to reset your Central Monitor when alerting.
8. **On/Off Switch** - Used to power on or off.
9. **Antenna Port** - Component not available at this time.
10. **Volume Control** - Increase or decrease the alert volume. "Mute" can only be used when using TL-4016P pager and system.
11. **AC Adapter Input** - Connect AC adapter here to power Central Monitor.
12. **Antenna** - Increases the signal strength from components.
13. **Hook for Mounting Bracket** - Slide bracket off bottom of monitor to use. When mounting on a wall, use hooks to hang on mounting bracket.
14. **Battery Compartment** - Batteries included.
15. **Base/Mounting Bracket** - Used as base when Central Monitor is sitting on a table or slide bracket off bottom of monitor to use as a wall mounting bracket.

# Setup

## Your System is Ready to Use

This monitor is a receiver and will automatically pick up alerts from the following:

- TL-4005HOME
- TL-4005SYS
- TL-4016
- 4017-CB

## Setting Up Your System

1. Plug AC adapter into TL-4015 Central Monitor and into power outlet.
2. Move the switch on the side of the Central Monitor to the "On" position.


Your monitor will be in ready mode and should display as follows




3. Select Volume desired.
4. Activate component. The Central Monitor LCD Display will audibly and visually display component activated.
5. Press "Review" button to silence/reset Central Monitor. \*Reset should be done at component.
6. Test all components to insure working as designed before putting into service.
7. Call Customer Service at 800.650.3637 with any questions. \*All components can have custom programmed #'s please contact your distributor or Smart for more information.

# Component Activated - Display on Central Monitor


Ready mode - No component activated:

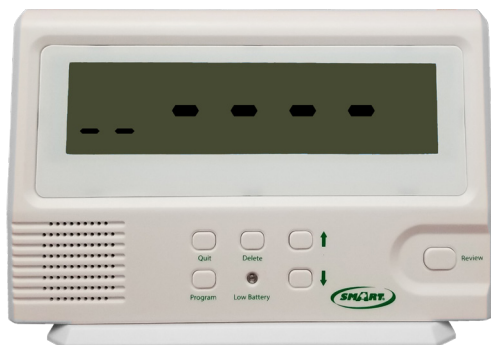

TL-4005SYS - Door System activated:

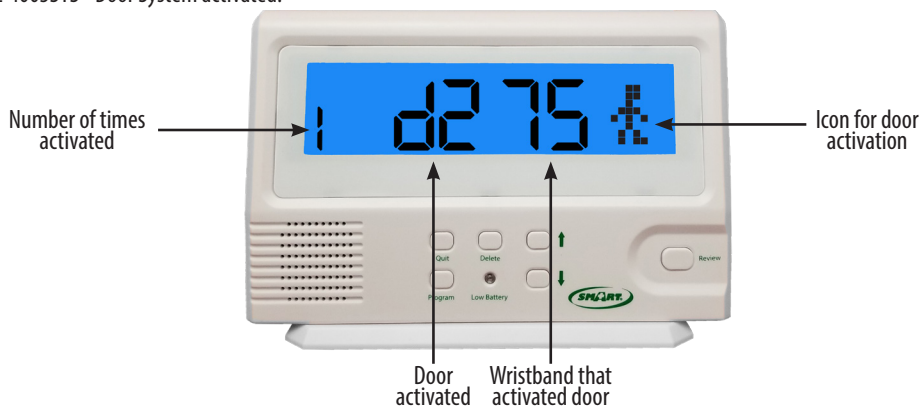


TL-4016 - Call Button pressed:

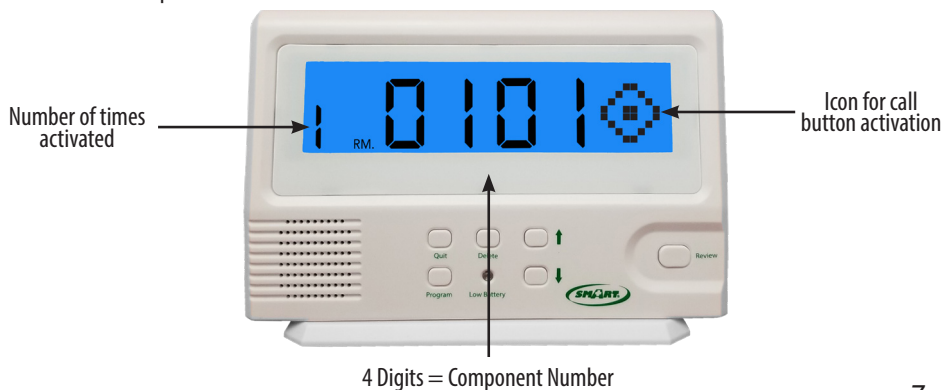


# Component Symbols


TL-4005SYS - Icon for Door System activation:

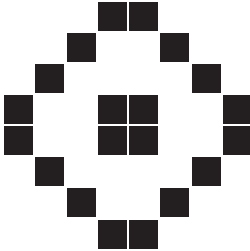


TL-4016 - Icon for Call Button activation:

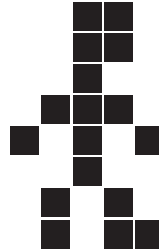


TL-4016 - Icon for Pressure Bed or Chair Pad activation:

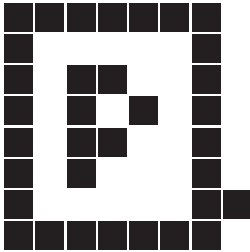


TL-4016 - Icon for Floor Mat activation:

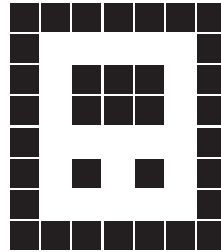

TL-4017 - Icon for Pull Cord activation:

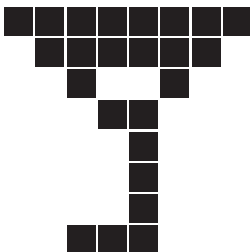


TL-4017 - Icon for Call Button activation:

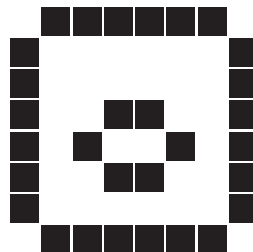



# ***Mounting The Monitor***

1. Slide mounting bracket off the bottom of the monitor
2. Mount bracket to the wall.
3. Slide hooks on back of CMU onto the mounting bracket

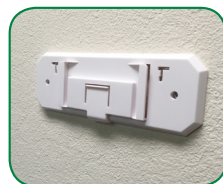

## ***Clear All Alerts***

### **To Clear All Alerts:**

Press and release "Delete" button.

## Range:

- The range between the TL-4015 Central Monitor and components is 150-300 feet maximum. However, environmental factors such as concrete or brick walls and heavy electronic equipment may affect range.

***If you need help, please feel free to contact us and our world-class customer service team will be happy to help. Call 800-650-3637.***

# Warning

**IMPORTANT INFORMATION:** Cordless and Wireless Systems rely on wireless technology which is subject to physical and environmental considerations. Please be aware that the transmitter will not be 100% accurate if it is out of range at any time. It is the end-user's responsibility to make sure that the product is used correctly and within range of each other. Cordless and wireless systems and devices are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all caregivers receive periodic training in the operation of these systems and that the devices are tested daily. It is important to read and follow the directions.

## **Read the instructions and Legal Disclaimer.**

The Smart Caregiver Corporation devices are designed to be installed by the end user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

**A.) Direct patient supervision**

**B.) Adequate training for staff personnel for fall prevention and elopement**

**C.) Testing of the system before each use**

**WARNING:** The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver.

Where permissible, it is the seller's intent to limit any liability for special, consequential or incidental economic damages to refund of the purchase price or replacement of the goods. Seller disclaims any liability for compensatory or other damages arising out of the use of the goods.

**Limited Warranty:** Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a warranty period from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty. No express or implied warranties are given and no affirmation of seller, by words or action will constitute a warranty. The goods being purchased are being sold on a "with all faults" basis. The entire risk as to the quality and performance of the goods is with the buyer. Should the goods prove defective following their purchase, the buyer, and not the manufacturer, distributor or retailer, assumes the entire cost of all necessary installation, servicing or repairs.



**Call Today**  
**(800) 650-3637**



We take pride in our outstanding customer service. At Smart Caregiver Corporation we strive to keep our customers 100% satisfied.

**Tel: (800) 650-3637 Fax (707) 781-7440**  
**[www.smartcaregiver.com](http://www.smartcaregiver.com)**

### **Notes**

- To ensure optimal wireless performance and reduce electrical interference, do not place the Central Monitor close to metal objects or other electronic devices.
- Do not expose the Central Monitor to heat or direct sunlight.
- Avoid dampness and water.